



The Fertility2Family Pregnancy Midstream Test is a self-test for the qualitative detection of human chorionic gonadotropin (hCG) in urine as an aid to early detection of pregnancy. This test does not confirm that a pregnancy is viable. Confirm results with a healthcare professional.


PLEASE READ ALL INFORMATION IN THIS LEAFLET BEFORE PERFORMING THE TEST

Pregnancy Midstream Test INSTRUCTIONS FOR USE

For self-testing. For in vitro diagnostic use only. (Not to be taken internally)

How to Perform a Pregnancy Test?


1. Remove the test stick from the foil wrapper. Do not use if expired, the pouch is open or damaged.
2. Remove the cap to expose the absorbent tip. Avoid contact with the absorbent tip.
3. Hold the stick by the thumb grip with the absorbent tip pointing downward. Urinate onto the absorbent tip until it is thoroughly wet. Do not urinate directly on the result window.
4. Recap the device and lay the test stick on a flat surface with the result window facing up.
5. Make sure the control line is present before reading your results. The test result should be read in 5 minutes; do not read it after 10 minutes.
6. Single use only. Discard after use.

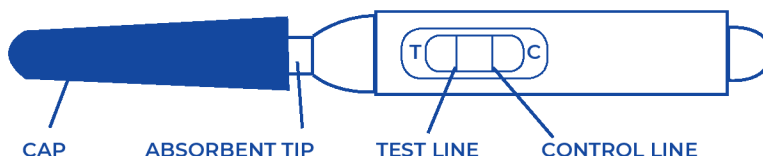

Interpreting the Test Results

A result is valid only if a coloured control (C) line appears.


Negative

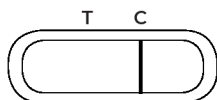

Negative:

If only one coloured line appears in the control (C) area and no line appears in the test (T) area, the result is negative. This means hCG was not detected at or above the test sensitivity at the time of testing. Testing too early or diluted urine can produce a negative result.



Positive

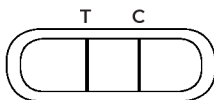

Positive:

If two coloured lines appear, one in the control (C) area and one in the test (T) area, the result is positive. Any visible test (T) line within the read time, even if faint, should be treated as positive. This means hCG has been detected. Confirm with a healthcare professional.

Invalid

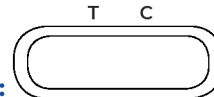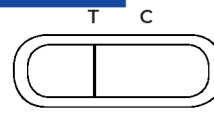

Invalid:

If no coloured line appears in the control (C) area, the result is invalid (even if a line appears in the test area). Retest with a new test stick. If the problem persists, contact the Australian sponsor.

FAQs

Q: When can I start testing?

A: You can test before your expected period. Testing early increases the chance of a negative result even if you are pregnant. If you get a negative result and your period is late, retest after 48–72 hours using first-morning urine or seek medical advice.

Q: What should I do if I get a positive result?

A: A positive result indicates hCG has been detected. Make an appointment with your doctor or healthcare professional to confirm the result and discuss next steps.

Q: What should I do if I get a negative result?

A: A negative result means hCG was not detected at or above the test sensitivity at the time of testing. If your period is late, retest after 48–72 hours using first-morning urine. Seek medical advice if your period remains late or symptoms change.

Q: The Test Line is not as dark as the Control Line. What does this mean?

A: Any visible test (T) line within the read time can indicate hCG has been detected. Line darkness does not measure how far along a pregnancy is. Confirm with a healthcare professional.

CONTENTS

- Each pouch contains one test stick and one desiccant. Do not ingest the desiccant.

STORAGE & STABILITY

- Store between 4°C and 25°C up to the expiry date.
- Keep Midstream Test in the sealed pouch until use.
- Protect from direct sunlight, moisture and heat.
- Do not freeze. Shelf life is 18 months.

WARNINGS & PRECAUTIONS

- As with all diagnostic tests, a diagnosis should not be based on the result of a single test. Confirm results with a healthcare professional and consider clinical findings.
- Read directions for use carefully before performing this test. Pay attention to the position of the test (T) and control (C) lines.
- Use immediately after opening the pouch. Prolonged exposure to ambient humidity can cause product deterioration.
- Treat urine samples and used devices as potentially infectious. Avoid contact.
- Dispose of the used test in household waste. Do not flush.
- If testing prior to the first day of a missed period, it is possible to detect a very early pregnancy that may not continue. This test does not determine pregnancy viability. If results are unexpected, retest and seek medical advice.
- Do not use beyond the labelled expiry date.
- Single-use only. Do not reuse. Discard after use.
- Do not touch the absorbent tip or the result window.
- Do not use if the pouch is damaged or open.
- Medications containing hCG can affect the result. Consult a healthcare professional if you are taking medicines that contain hCG.
- Discard desiccant safely. Do not ingest. Keep out of reach of children.
- Due to the high sensitivity of this test (25 mIU/mL), detection of hCG in the urine of a peri-menopausal or menopausal woman may not indicate pregnancy and may represent pituitary hCG. Laboratory confirmation may be required.

Sensitivity to hCG - 25 mIU/mL

Single Use



Store Between



AUSTRALIAN SPONSOR
Fertility2Family ABN: 54 063 506 152
PO Box 72, Glenorchy, Tasmania 7010
www.fertility2family.com.au
info@fertility2family.com.au

MANUFACTURER
Nantong Egens Biotechnology Co., Ltd
Block A No.15 Building No. 1692 Xinghu
Avenue, Nantong Economy & Technology
Development Zone, 226010, China